


# Listen-In Operation Manual

## Establish a Bluetooth Connection

- Enable Bluetooth on Device
- Open Listen-In Application
- If desired Stethoscope is not available on the Dashboard Activity
  - Pair Stethoscope with Application
    - Go to the Menu and select “Pair” on the Stethoscope
    - Press the “Discovery” button while Stethoscope is trying to pair 
    - When Stethoscope is discovered, press and hold Stethoscope name to open options in application

✓ DONE

Options

RENAME


PAIR

- Select “PAIR” and enter the pairing code displayed on Stethoscope


- Connect Stethoscope to Application
  - Go to the Menu and select “Connect” on the Stethoscope
  - Press the “Connect” button while Stethoscope is trying to connect



## Record an Audio Sample

- Establish a Bluetooth Connection
- When a connection is made the Phonocardiogram will begin plotting data
- Press the Phonocardiogram to go to a full screen version
- Press the “RECORD” button on the top menu 
- When you are finished recording select the “STOP RECORDING” button
- Name your audio sample

## Listen to an Audio Sample

- Establish a Bluetooth Connection
- From Phonocardiogram full screen, select “Audio Player” button from top menu 
- Choose desired audio file from spinner, and press the play button
- Control the audio using the “+” and “-” buttons on the Stethoscope

## Sharing an Audio Sample

- Establish a Bluetooth Connection
- Select the audio sample you want to share

**SHARE...**

- Press the “Share...” button from top menu
- Hit “Yes” when asked if you want to upload the file
- Select the Social Media service you wish to share this sample via
- Include a comment describing this audio sample and submit the comment/tweet